



# COOL TREATMENTS

BY ASCLEPION



**BRANDED PROTOCOLS**

powered by **Dermablade**



INTRODUCING  
**COOL LASER**  
powered by **Dermablade**

---

# HOW COOL CAN A LASER BE?

---

## WHAT MAKES A LASER COOL?

Is it the way it treats the skin? The results it delivers?

Or is it simply because it's from Asclepion?

The answer is: all of the above!

**COOL LASER** is the next-generation Erbium:YAG technology that redefines skin resurfacing. And at the heart of it all? Dermablade, Asclepion's gold standard Erbium:YAG laser, trusted by professionals worldwide.

It's **COOL** because it features cold ablation, removing tissue with extreme precision while keeping the surrounding skin happy and undisturbed.

It's **COOL** because it enables a range of game-changing treatments, from ultra-precise eyelid rejuvenation to full-face resurfacing.

And most importantly, it's **COOL** because it's made by Asclepion, the leader in high-performance medical laser technology.

---

# WHAT ARE COOL TREATMENTS?

---

## A NEW STANDARD IN LASER PROTOCOLS

Asclepion introduces a new era of **branded laser protocols**, designed to unlock the full potential of the Dermablade Erbium:YAG laser.

Whether for lifting, resurfacing, or rejuvenation, each treatment is designed for precision, reliability, and superior clinical outcome.

## WHY COOL TREATMENTS WORK

### OPTIMIZED PARAMETERS

Each protocol is based on carefully calibrated fluence, pulse duration, and scanning techniques for maximum effectiveness.

### MULTIPLE HANDPIECES

Integrated in each treatment session, ensure targeted, controlled, and effective energy application.

### DUAL THERAPY APPROACH

Combining two steps and multiple laser technologies to enhance treatment efficacy and expand treatment possibilities.



---

# YOUR GUIDE TO COOL TREATMENTS

---

## PREMIUM TREATMENT CONCEPTS

Exclusive to Dermablade users, COOL TREATMENTS offer medical professionals premium, branded laser procedures that set new benchmarks in safety and effectiveness.

Standardized yet adaptable, each protocol follows clinically tested guidelines while allowing flexibility to tailor treatments to individual patient needs. The result?

**Consistent, high-quality outcomes with minimal downtime.**

## HERE'S WHAT'S COOL ON THE MENU:

### ★ COOL BLEPH

Say goodbye to tired eyes! A non-surgical blepharoplasty for upper and lower eyelids, delivering a refreshed, youthful look.

### ★ COOL ORA

Targeting perioral wrinkles with precision, this resurfacing treatment restores smooth, firm skin around the mouth.

### ★ COOL YELLOW


A powerful dual-wavelength treatment combining vascular therapy with skin resurfacing for a clearer, more even complexion.

### ★ COOL VIVE

Neck aging? Not on our watch! This dual-step laser treatment tightens and firms sagging skin in the neck area.

# COOLBLEPH

BY ASCLEPION

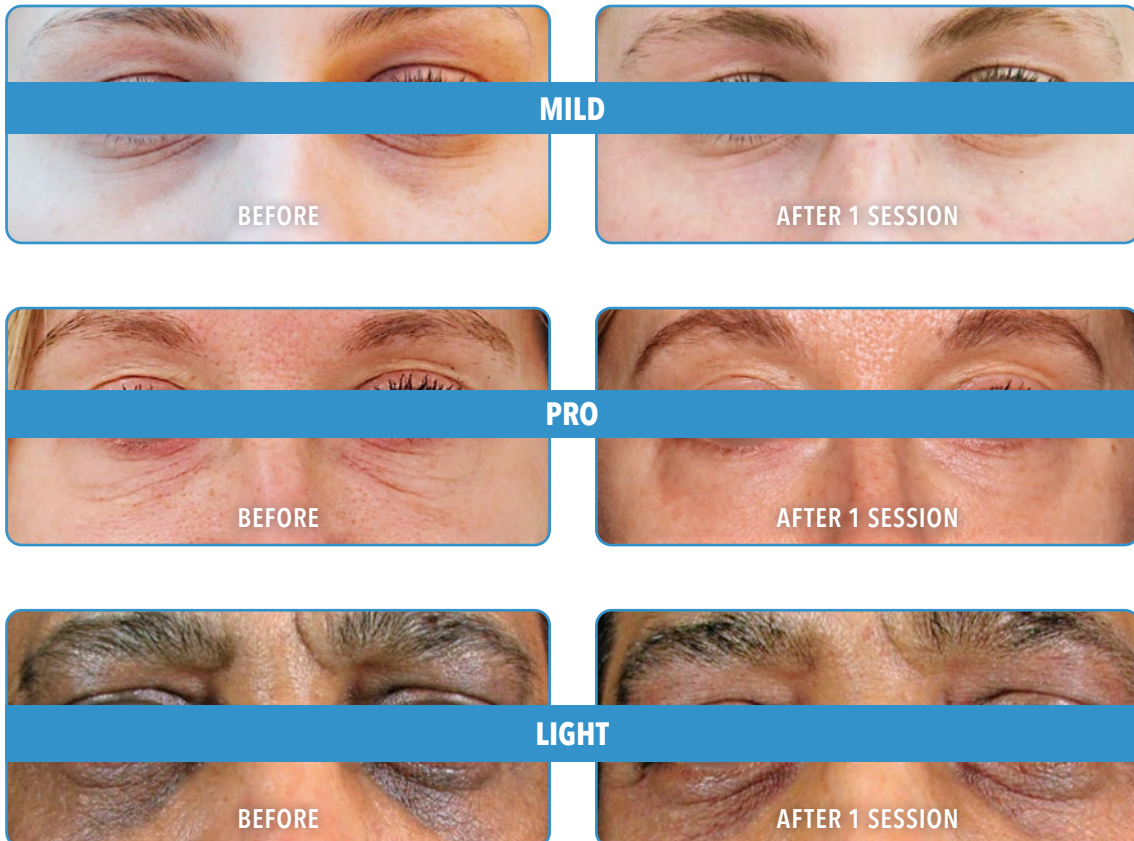

Pictures by Courtesy of Dr. Tapan Patel | PHI Clinic | London

# ..... NON SURGICAL BLEPHAROPLASTY .....

## TREATMENT CONCEPT

COOL BLEPH is a non-invasive laser treatment for tightening and rejuvenating the upper and lower eyelid skin. It smooths wrinkles, reduces sagging, and brightens pigmentation for a youthful, refreshed appearance without surgery. Available in three versions (Mild, Pro, Light) to suit different patient needs.

## BENEFITS

MILD: Smooths skin and evens tone

PRO: Reduces lines and wrinkles by up to 60% in one session

LIGHT: Improves discoloration in 1-3 sessions

## PROTOCOLS

|                     | MILD                  | PRO                  | LIGHT              |
|---------------------|-----------------------|----------------------|--------------------|
| Step 1 – Dermablate | VarioSpot 4mm         | VarioSpot 4mm        | VarioSpot 4mm      |
| Step 2 – Dermablate | MicroSpot Expert 10%  | MicroSpot Expert 10% | MicroSpot COLD 10% |
| Numbing             | No                    | Yes                  | No                 |
| Treatment time      | 15 min                | 15 min               | 15 min             |
| Recommended course  | 2 sessions            | 2 sessions           | 2-3 sessions       |
| Recovery            | 3 days light swelling | 4 days swelling      | 3-6 days swelling  |

# COOLORA

BY ASCLEPION



Pictures by Courtesy of Dr. Tapan Patel | PHI Clinic | London

# ..... PERIORAL RESURFACING .....

## TREATMENT CONCEPT

COOL ORA is a specialized resurfacing treatment that restores smoothness and firmness to the perioral area. By precisely targeting wrinkles and skin laxity, it removes aged elastotic tissue, stimulating natural regeneration and tightening. Ideal for those seeking a rejuvenated mouth contour.

## BENEFITS

Reduces lines and wrinkles for a smoother perioral area

Tightens skin and restores elasticity

Over 60% improvement in one session

## PROTOCOLS

|                     |                                                             |
|---------------------|-------------------------------------------------------------|
| Step 1 – Dermablade | VarioSpot 4mm                                               |
| Step 2 – Dermablade | MicroSpot EXPERT 10% perioral<br>MicroSpot COLD 10% on lips |
| Numbing             | Yes                                                         |
| Treatment time      | 20 min                                                      |
| Recommended course  | 1-2 sessions                                                |
| Recovery            | 5-7 days light swelling and scabbing                        |

# COOL YELLOW

BY ASCLEPION



Pictures by Courtesy of Mrs. Sarah Bow | Asclepion UK | London

# DUAL VASCULAR & RESURFACING TREATMENT

## TREATMENT CONCEPT

COOL YELLOW is an advanced dual-wavelength laser treatment that combines 577nm yellow light for vascular concerns with Er:YAG resurfacing for pigmentation and textural improvement. This synergy delivers comprehensive skin renewal with minimal downtime.

## BENEFITS

Reduces redness, vascular lesions, and solar damage

Up to 40% improvement in one session


Minimal downtime

## PROTOCOLS

|                                |                                                                                    |
|--------------------------------|------------------------------------------------------------------------------------|
| Step 1<br>QuadroStarPRO YELLOW | Scanner 60% density on diffuse redness<br>Pen 1mm on singular vessels              |
| Step 2<br>Dermablate           | MicroSpot COLD 10% for mild peeling<br>MicroSpot EXPERT 10% for medium resurfacing |
| Numbing                        | No                                                                                 |
| Treatment time                 | Up to 30 min                                                                       |
| Recommended course             | 2-4 sessions                                                                       |
| Recovery                       | 2-7 days (face) / 14-20 days of peeling (body) according to laser settings         |

# COOLVIVE

BY ASCLEPION

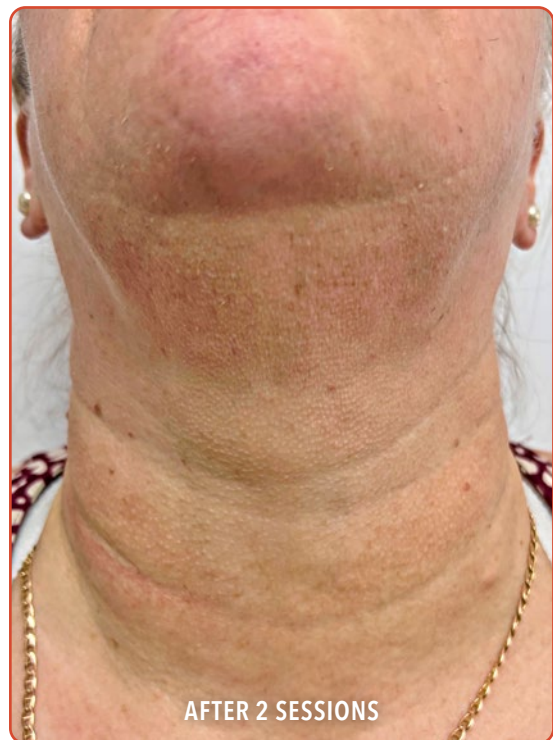

Pictures by Courtesy of Skiin Laser Clinic | Mrs. Kerry Hutson | London

# ..... DUAL NECK REJUVENATION .....

## TREATMENT CONCEPT

COOL VIVE is a targeted laser lifting procedure designed to tighten and revitalize the neck area. By stimulating collagen production and enhancing elasticity, it restores definition and smoothness, offering a non-surgical alternative to neck lifts.

## BENEFITS

Firms skin and improves elasticity

Reduces wrinkles and solar damage

Provides a natural lifting effect

## PROTOCOLS

|                     |                      |
|---------------------|----------------------|
| Step 1 – Dermablade | MicroSpot EXPERT 10% |
| Step 2 – Dermablade | MicroSpot HOT 25%    |
| Numbing             | No                   |
| Treatment time      | 15 min               |
| Recommended course  | 2–3 sessions         |
| Recovery            | 48 hours             |

---

# 30 YEARS OF ERBIUM:YAG EXCELLENCE

---

## PIONEERING LASER INNOVATION SINCE 1995

For three decades, Asclepion has been at the forefront of Erbium:YAG laser technology, revolutionizing skin resurfacing, scar treatment, and aesthetic medicine.

- 1995** MCL29 Dermablade – Asclepion's first Er:YAG laser sets a new standard in dermatology
- 2007** Introduction of MicroSpot technology for fractional therapy
- 2015** Launch of Juliet – the feminine laser for women's intimate health
- 2018** Introduction of Romeo for anti-snoring laser therapy
- 2021** Launch of the next-generation Dermablade system
- Today** COOL TREATMENTS integrate 30 years of expertise for modern aesthetic needs

**30 YEARS OF EXPERTISE.**

The future of laser medicine starts here.





# COOLTREATMENTS

BY ASCLEPION

[www.asclepion.com](http://www.asclepion.com)

