



COOL TREATMENTS[®]

— RELOADED —

THE NEW STANDARD FOR
FIRST-CLASS LASER PROTOCOLS



by ASCLEPION LASER TECHNOLOGIES



INTRODUCING

COOL LASER[®]

powered by **Dermablade[®]**

COOL LASER[®]

Dermablade



HOW COOL CAN A LASER BE?

What makes a laser COOL?

Is it the way it treats the skin? The results it delivers?

Or is it simply because it's from Asclepion?

The answer is: all of the above!

COOL LASER® is the next-generation Erbium:YAG technology that redefines skin resurfacing. And at the heart of it all? Dermablate®, Asclepion's gold standard Erbium:YAG laser, trusted by professionals worldwide.

It's **COOL** because it features cold ablation, removing tissue with extreme precision while keeping the surrounding skin happy and undisturbed.

It's **COOL** because it enables a range of game-changing treatments, from ultra-precise eyelid rejuvenation to full-face resurfacing.

And most importantly, it's **COOL** because it's made by Asclepion, the leader in high-performance medical laser technology.

WHAT ARE COOL TREATMENTS®?

A new standard in laser protocols

Asclepion introduces a new era of **branded laser protocols**, designed to unlock the full potential of the Dermablade® Erbium:YAG laser.

Whether for lifting, resurfacing, or rejuvenation, each treatment is designed for precision, reliability, and superior clinical outcomes.

Why COOL TREATMENTS® work

OPTIMIZED PARAMETERS

Each protocol is based on carefully calibrated fluence, pulse duration, and scanning techniques for maximum effectiveness.

MULTIPLE HANDPIECES

Integrated in each treatment session, they ensure targeted, controlled, and effective energy application.

DUAL THERAPY APPROACH

Combining two steps and multiple laser technologies to enhance treatment efficacy and expand treatment possibilities.



YOUR GUIDE TO COOL TREATMENTS®

Premium treatment concepts

Exclusive to Dermablade® users, COOL TREATMENTS® offer medical professionals premium, branded laser procedures that set new benchmarks in safety and effectiveness.

Standardized yet adaptable, each protocol follows clinically tested guidelines while allowing flexibility to tailor treatments to individual patient needs. The result?

Consistent, high-quality outcomes with minimal downtime.

Just select. Treat. Stay COOL.

COOL TREATMENTS® are seamlessly integrated into Dermablade® for an effortless workflow.

Select the protocol directly on the treatment screen and let the system guide you with optimized parameters for safe, consistent results.

Request your update now and unlock the full COOL experience with your COOL Laser® Dermablade®.



HERE IS WHAT'S COOL ON THE MENU

The original COOL TREATMENTS®

This is where the COOL concept began.

Four signature laser protocols that established the foundation of the COOL approach: standardized treatments designed to address key aesthetic concerns with precision and reliability.

From eyelid rejuvenation and perioral resurfacing to vascular and neck treatments, these protocols define the original COOL philosophy:

Effective procedures, clear parameters, and minimal downtime.

★ COOL BLEPH

Say goodbye to tired eyes! A non-surgical blepharoplasty for upper and lower eyelids, delivering a refreshed, youthful look.

★ COOLORA®

Targeting perioral wrinkles with precision, this resurfacing treatment restores smooth, firm skin around the mouth.

★ COOL YELLOW®

A powerful dual-wavelength treatment combining vascular therapy with skin resurfacing for a clearer, more even complexion.

★ COOL VIVE®

Neck aging? Not on our watch! This dual-step laser treatment tightens and firms sagging skin in the neck area.

HERE IS WHAT'S **NEW** ON THE MENU

Fresh additions to the COOL lineup are here!

The COOL portfolio continues to expand.

Building on the success of the original protocols, three new treatments introduce additional options for addressing pigmentation, skin radiance, and early signs of aging.

By combining new laser technologies with refined treatment strategies, they expand the scope of COOLTREATMENTS® while staying true to the same principle:

Visible results with minimal downtime.

★ COOLGLOW®

A gentle yet effective superficial peeling designed for those who want refreshed, radiant skin with minimal downtime.

★ COOL PICO®

A dual-wavelength protocol combining the precision of a picosecond laser in targeting unwanted pigmentation with gentle fractional Er:YAG resurfacing to refine skin texture.

★ COOL RESET®

Turn back time in just one session by targeting the two areas that most often reveal your age: an advanced laser protocol tailored for hands and décolleté.



COOLBLEPH

BY ASCLEPION

NON SURGICAL BLEPHAROPLASTY

Treatment Concept


COOL BLEPH is a non-invasive laser treatment for tightening and rejuvenating the upper and lower eyelid skin. It smooths wrinkles, reduces sagging, and brightens pigmentation for a youthful, refreshed appearance without surgery. Available in three versions (Mild, Pro, Light) to suit different patient needs.

Benefits

MILD: Smooths skin and evens tone

PRO: Reduces lines and wrinkles by up to 60% in one session

LIGHT: Improves discoloration in 1-3 sessions

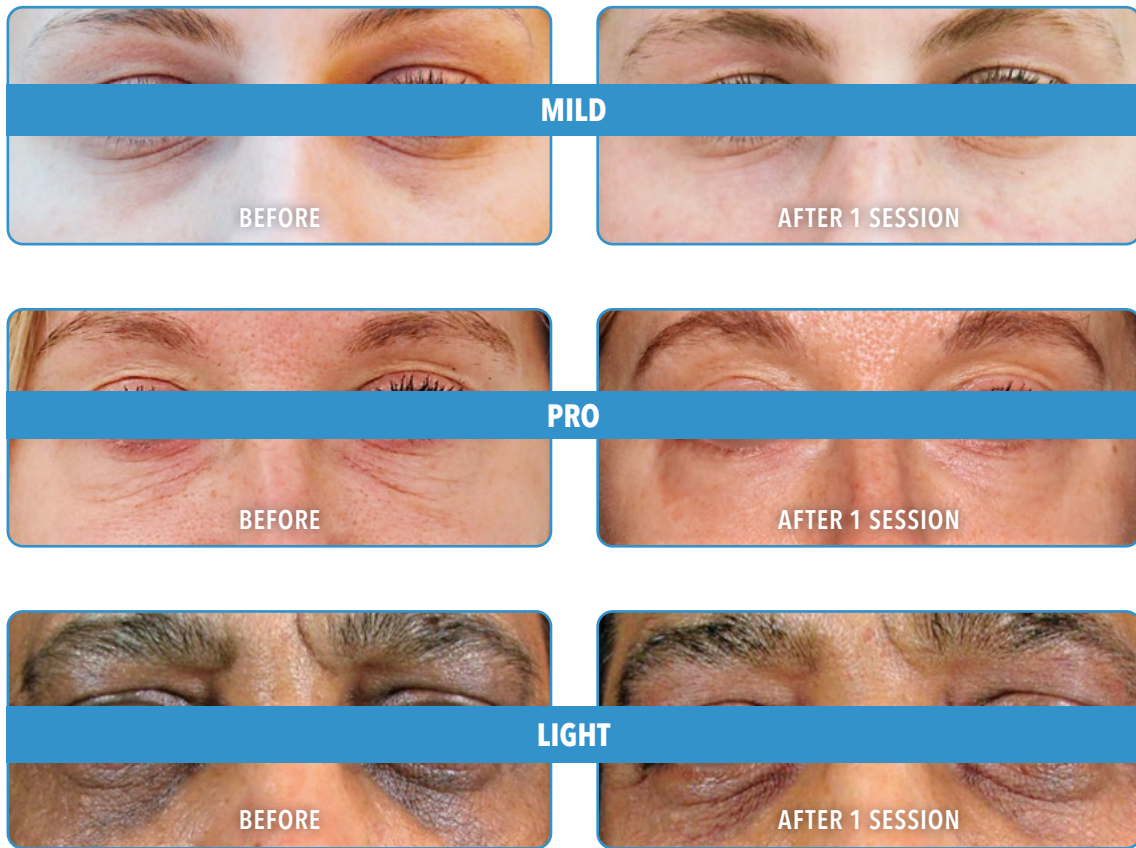

Pictures by Courtesy of Dr. Tapan Patel | PHI Clinic | London

Protocols

	MILD	PRO	LIGHT
Step 1 - Dermablate	VarioSpot 4mm	VarioSpot 4mm	VarioSpot 4mm
Step 2 - Dermablate	MicroSpot EXPERT 10%	MicroSpot EXPERT 10%	MicroSpot COLD 10%
Numbing	No	Yes	No
Treatment time	15 min	15 min	15 min
Recommended course	2 sessions	2 sessions	2-3 sessions
Recovery	3 days light swelling	4 days swelling	3-6 days swelling



COOLORA[®]

BY ASCLEPION

PERIORAL RESURFACING

Treatment Concept

COOL ORA[®] is a specialized resurfacing treatment that restores smoothness and firmness to the perioral area. By precisely targeting wrinkles and skin laxity, it removes aged elastotic tissue, stimulating natural regeneration and tightening. Ideal for those seeking a rejuvenated mouth contour.

Benefits

Reduces lines and wrinkles for a smoother perioral area

Tightens skin and restores elasticity

Over 60% improvement in one session



Pictures by Courtesy of Dr. Tapan Patel | PHI Clinic | London

Protocols

Step 1 - Dermablade	VarioSpot 4mm
Step 2 - Dermablade	MicroSpot EXPERT 10% perioral MicroSpot COLD 10% on lips
Numbing	Yes
Treatment time	20 min
Recommended course	1-2 sessions
Recovery	5-7 days light swelling and scabbing



COOL YELLOW[®]

BY ASCLEPION

DUAL VASCULAR & RESURFACING TREATMENT

Treatment Concept

COOL YELLOW[®] is an advanced dual-wavelength laser treatment that combines the 577 nm yellow wavelength of the QuadroStarPRO YELLOW with Er:YAG resurfacing. It targets vascular and pigmentation concerns while improving overall skin texture with minimal downtime.

Benefits

Reduces redness, vascular lesions, and solar damage

Up to 40% improvement in one session

Minimal downtime



Pictures by Courtesy of Mrs. Sarah (McNulty) Bow | Asclepion UK | London

Protocols

Step 1 - QuadroStarPRO YELLOW	Scanner 60% density on diffuse redness Pen 1mm on singular vessels
Step 2 - Dermablate	MicroSpot COLD 10% for mild peeling MicroSpot EXPERT 10% for medium resurfacing
Numbing	No
Treatment time	Up to 30 min
Recommended course	2-4 sessions
Recovery	2-7 days (face) 14-20 days of peeling (body) according to laser settings



COOL VIVE®

BY ASCLEPION

DUAL NECK REJUVENATION

Treatment Concept


COOL VIVE® is a targeted laser lifting procedure designed to tighten and revitalize the neck area. By stimulating collagen production and enhancing elasticity, it restores definition and smoothness, offering a non-surgical alternative to neck lifts.

Benefits

Firms skin and improves elasticity

Reduces wrinkles and solar damage

Provides a natural lifting effect

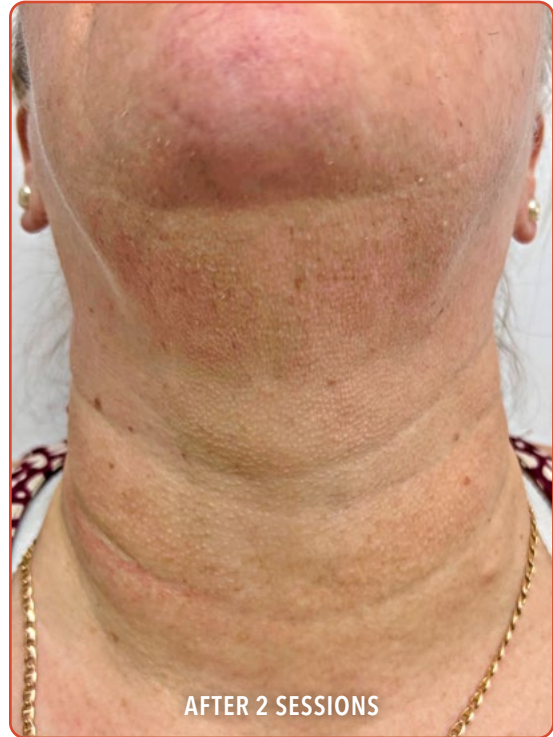

Pictures by Courtesy of Skiin Laser Clinic | Peterborough | UK

Protocols

Step 1 - Dermablade	MicroSpot EXPERT 10%
---------------------	----------------------

Step 2 - Dermablade	MicroSpot HOT 25%
---------------------	-------------------

Numbing	No
---------	----

Treatment time	15 min
----------------	--------

Recommended course	2-3 sessions
--------------------	--------------

Recovery	48 hours
----------	----------



COOLGLOW[®]

BY ASCLEPION

LUNCH BREAK PEELING

Treatment Concept

A gentle yet effective superficial peeling designed to refresh the skin and enhance its natural radiance with minimal downtime.





The treatment addresses the epidermal layer, creating precise micro-ablated zones that support visible skin renewal.

Benefits

Improves skin brightness and overall tone

Smooths the skin surface and refines texture

Creates a fresh, even-looking complexion with no interruption to daily activities

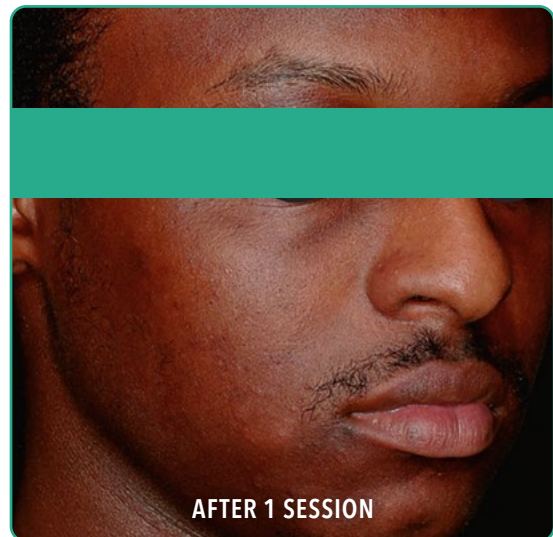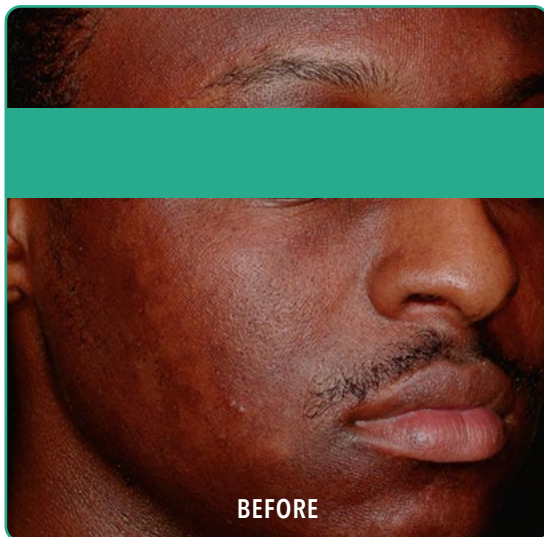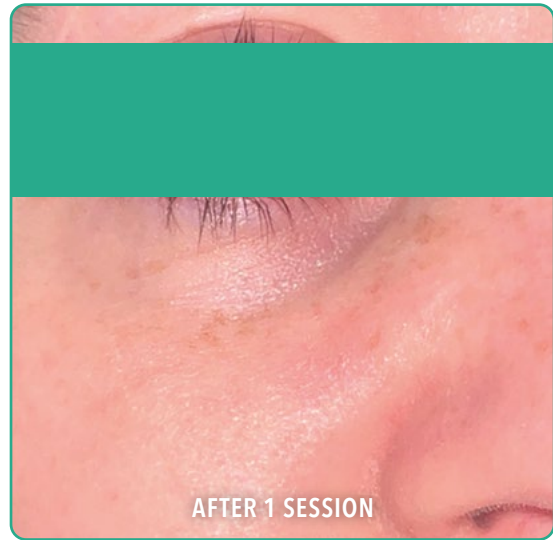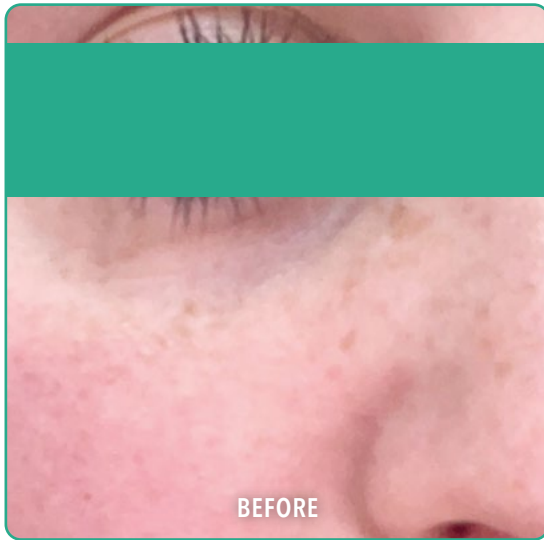

Pictures by Courtesy of Mrs. Sarah (McNulty) Bow | Asclepion UK | London

Protocols

Step 1 - Dermablate	MicroSpot COLD 10%
Numbing	No
Treatment time	15 min
Recommended course	3 sessions
Recovery	Mild sunburn-like sensation for 4-6 hours



 **COOLPICO[®]**

BY ASCLEPION

ADVANCED PIGMENT CORRECT & SKIN RENEWAL

Treatment Concept

This dual-wavelength protocol combines the high precision of the PicoStar[®] picosecond laser for the targeted treatment of unwanted pigmentation with gentle fractional Er:YAG resurfacing to refine skin texture and enhance radiance, leaving the skin visibly brighter and more even.

Benefits

Reduces unwanted pigmentation

Refines skin texture

Reveals a clearer, more luminous complexion



Pictures by Courtesy of Dr. Bárbara Meléndez | Spain

Protocols

Step 1 - PicoStar	Zoom FT or Flat-Top 532 HP, 5 mm
Step 2 - Dermablade	MicroSpot COLD 10% for mild peeling MicroSpot EXPERT 10% for medium resurfacing
Numbing	No
Treatment time	30 min
Recommended course	2 sessions
Recovery	7-10 days temporary darkening of pigment, 4-5 days peeling



★ COOL RESET[®]

BY ASCLEPION

SIGNATURE REJUVENATION FOR HANDS & DÉCOLLETÉ

Treatment Concept

Turn back time in just one session by targeting the two areas that most often reveal your age, delivering visible renewal. This advanced dual-step laser protocol smooths fine lines, refines texture, and restores a youthful glow, leaving hands and décolleté firmer, brighter, and beautifully refreshed.

Benefits

Reduces pigmentation and sun damage

Smooths fine lines and improves skin texture

Restores brightness for visibly younger-looking skin



Pictures by Courtesy of Skiin Laser Clinic | Peterborough | UK

Protocols

Step 1 - Dermablade	MicroSpot EXPERT 10% on the whole décolleté area MicroSpot HOT 25% on the décolleté V-shaped area
Step 2 - Dermablade	MicroSpot EXPERT 10% on hands, for skin types I-III MicroSpot COLD 10% on hands, for skin types IV-VI VarioSpot 3-4 mm only in case of keratosis
Numbing	No
Treatment time	20 min
Recommended course	2 sessions
Recovery	7-10 days of dry skin barrier and peeling

THE POWER OF COOL SYNERGIES

Where laser combinations unlock new treatment dimensions

Dermablate® COOL Laser® sets the benchmark for precise Er:YAG resurfacing. But its true strength emerges when combined with complementary laser technologies.

Together with QuadroStarPRO YELLOW and PicoStar®, Dermablate® expands its clinical capabilities, allowing physicians to address vascular lesions, pigmentation, skin texture, and overall skin quality with greater precision and flexibility.

Perfectly coordinated protocols unlock a wider range of indications and enable physicians to deliver outstanding results with efficiency and minimal downtime.

The COOL Laser® is just the beginning.
The full COOL combination takes your treatments further.



THE ERBIUM:YAG LEGACY

Pioneering laser innovation since 1995

For three decades, Asclepion has been at the forefront of Erbium:YAG laser technology, revolutionizing skin resurfacing, scar treatment, and aesthetic medicine.

1995 MCL29 Dermablade – Asclepion’s first Er:YAG laser sets a new standard in dermatology

2007 Introduction of MicroSpot technology for fractional therapy

2015 Launch of Juliet – the feminine laser for women’s intimate health

2018 Introduction of Romeo for laser-based snoring therapy

2021 Launch of the next-generation Dermablade® system

Today COOLTREATMENTS® bring together 30 years of expertise for modern aesthetic needs




30 YEARS OF EXPERTISE.

The future of laser medicine starts here.

COOL LASER[®]
BY ASCLEPION



COOLTREATMENTS[®]
— RELOADED —

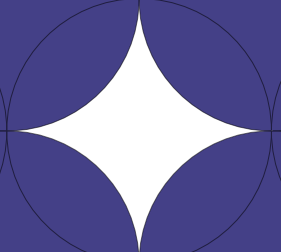

www.cooltreatments.com

Asclepion
Laser Technologies